

# FootActive

## Kids

Orthotic insoles



For flat feet and fallen arches

Helps relieve children's heel pain

## Support for little feet

- ✓ 3/4-length medical-grade orthotic developed by Australian podiatrists
- ✓ Corrects over-pronation, supports fallen arches and flat feet
- ✓ Fits all types of children's footwear including school and sport shoes
- ✓ Strong arch support + deep heel cup
- ✓ Shock-absorbing Durapontex heel pad

FootActive *Kids* orthotics provide excellent biomechanical support and re-alignment, thereby helping to restore your child's natural walking pattern. By re-aligning the child's feet early on, it will allow the feet to develop more naturally and many biomechanical complaints can be prevented or the impact lessened.

FootActive *Kids* are useful for children with flat feet, fallen arches or *over-pronation* (rolling in of the foot). This insole also provides relief for *Sever's Disease* (children's heel pain) and growing pains, including achings legs and knee pain (*Osgood Schlatter's*).

FootActive *Kids* orthotics are safe to use, however, if your child suffers from a severe foot deformity or particular illness or medical condition, we recommend you consult your GP or specialist first, before using FootActive.



### Fitting and Care

Place the insoles right to the back of the shoe. It may take a few days to get used to the orthotics. If they feel uncomfortable, please only wear the insoles for 1-2 hours a day, until they feel comfortable. The insoles can be washed by hand with a wet, soapy cloth. Please do not soak them and please do not put them in the washing machine or dryer!